

ALDEHYDE FREE





TM



About us...

Genex Hygiene has an rich experience of disinfection products. We are in hospital and allied sterilsation consumable products. Our manufacturing facility is WHO, GMP, GLP, FDA, ISO 9001-2008 certified.

Our product are research & developed in our R&D facility. We have take all stringent steps for manufacturing which assure you our QUALITY products. All product are tested and certified at reputed laboratories.

Enviroguard is a balanced stabilized blend of 3rd generation twin chain quaternary ammonium compounds. Enviroguard is recommended for use as a hard surface disinfectant in hospitals, nursing homes, other various type of medical facilities, Food Industries, Pharme Industries and Laboratories. It delivers 99.9999% kill of more than 148 types of different pathogens including HIV, HEV, HCV, Polio, H1N1, H5N1, etc with no evidence of resistance. Eliminating the need to rotate disinfectants to avoid resistance buildup.

Area of Application

- Surface Environment Discarding j
- Walls Instrument Disinfection Terminal Disinfection



Effective in Critical Areas

All High Risk Areas
Operation Theaters
ICUs
Cath Labs

• Burn Units • Dental Clinics • Blood Banks • Laboratories • Dialysis Units

Feature & Benefits

- Highly Effective Bactericidal, Fungicidal, Virucidal Non Corrosive
 - Non-Toxic User Friendly Easy to Use Cost Effective
- Eco Friendly Bio-Degradable Contact Time upto 10 Mins. Pleasant Odour

Comparison of Available Molecules in the Market





eros

Content

3rd GenerationQuaternary Ammonium CompoundAlkyl Dimethyl Benzyl Ammonium chloride2.37%Alkyl Dimethyl Ethylbenzyl Ammonium chloride2.37%Inert Ingredients95.26%

(US-EPA certified formulation)

Direction of Use <u>Dilution :</u> Critical Area- 1:64 (15 ml in 1 Litre of water) Non-Critical Area- 1:128 (7.5 ml in 1 Litre of water)

Surface Cleaning :

Critical Area - Mix 15 ml Enviroguard in 1 litre of water for mopping/wiping floors, walls, furnitures, equipments, O.T. Table, O.T. Labs etc.

Non-Critical Area- Mix 7.5ml Enviroguard in 1 litre of water for mopping waiting halls, wards, corridors, OPD, reception area etc.

Fogging Dilution :

(1:64) Add 15ml of Enviroguard with 1 Litre of water for every 1000 cu. ft. in high risk areas like OT, ICU, ICCU, NCU, Cath Lab, AKD etc.

Re-Entry Level in O.T. After Fogging -40 Minutes (Till the Mist Settles)

Packaging : 100 ml, 1 Liter Concentrate

Shelf Life : 3 Years in packed condition 64 days for diluted solutions

> **Genex Hygiene**(A unit of a GenX Hygiene PV Ltd) 5A/114, Mittal Industrial Estate, Andheri-Kurla Road, Andheri(East), Mumbai – 400059 genexhygiene@gmail.com | www.ganexhygiene.com 9315023115, 9315048717

be spra

with

Envirofog