

Instrugen™ spray

Ready-to-Use Instrument Lubricant and Rust Inhibitor

About Instrugen Spray

Instrugen Spray is Non-silicone, patient-safe lubricant that keeps box locks working freely and maintains effectiveness of grasping and cutting surfaces. Protects all types of metal instruments, including carbon steel. Instrugen Spray is a ready-to-use oil lubricant that protects surgical instruments by forming a protective coating that prevents rust, staining and spotting while restoring articulation to sticky box locks and joints. Used routinely will help reduce repair and replacement cost of expensive instrumentation.

It is a non-sticky, non-toxic, silicone free and water-soluble mineral oil lubricant that can withstand the high temperatures of autoclaving without becoming gummy.

Composition : Light Liquid Paraffin(Medicinal grade)

Need of lubrication for surgical Instruments??

Protection : Helps protect surgical instruments, which represents a large investment to health care facilities.

Compatibility : Its neutral pH is safe for use on stainless steel or aluminum

Cost-Savings : Helps extend instrument life and facilitates future optimal use
A lubricant spray is advised. Sprays are safer, cost less and take up less counter space.

Features

- High-grade Medical grade oil lubricant and rust inhibitor
- Prevents rust, spots, stains and corrosion prolonging the life of the instrument
- Prevents damage due to the misuse of stiff or grinding instruments reducing repair and replacement costs
- Steam penetrable, steam sterilizable and ETO sterilizable
- Non-oily, non-sticky, non-toxic and silicone free

Direction for use :

For Spray Applications:

Prior to Sterilization:

1. Spray Instrugen Spray over surface of instruments and into box locks and joints. Hold the spray can as vertically as possible.
2. Allow instruments to set for 30 seconds.
3. Test box locks and joints for stickiness or grinding. Re-apply if necessary. (Seized instruments may require treatment by immersion in a bath of Instrugen Spray.)
4. Instruments are ready for packing and sterilization. Do not rinse or wipe dry.



Product Specification

Colour : Colourless

Fragrance : Odourless

pH : Neutral

Shelf life : 4 years

Solubility : Water miscible

Storage : Store at room temperature

Shake well before use

Pack size : 100 ml & 500 ml