

Rustclean-IR™

High Efficiency Rust Spots, Stains Remover

About Rustkleen-IR

Rustkleen-IR is a ready to use high efficiency rust remover with a fast action formula. It help in removing and converting rust into iron and further protecting it from oxidation. The quick action of Rustkleen-IR leaves the treated surface in a relatively passivated condition. Rustkleen-IR restores original finish and increase use-life of instruments.



Feature & Benefits

Mild acidic solution removes rust, stains, spotting and corrosion. Restores instrument's original finish. Non-corrosive; will not harm stainless steel. Save money by restoring instruments instead of replacing or repairing them. Routine use of Rustkleen-IR will enhance the life and efficiency of stainless steel instruments. Restores articulation to box-locks and joints. Prevents destructive pitting on ridges of micro and ophthalmic instruments. Makes instruments look like new

Direction to use :

Cleaning in immersion tray

- Immerse instruments in warm 1-10% Rustkleen-IR solution (10-100 ml/litre, max 50 C)
- Instruments with carbide insert, use only 1-3% Rustkleen-IR solution (10-30 ml/litre, max 50C)
- After maximum. contact of approximately 1 hour, remove instruments & rinse thoroughly with water & dry

Cleaning in Ultrasonic bath

- Immerse instruments in warm 1-2% Rustkleen-IR solution (10-20 ml/litre, max 50 C)
- Contact time of approx. 1-5 min in accordance with instrument manufacture's data.

Composition

Phosphoric Acid, Non Ionic Surfactants & Glycol ether butyl

Appearance form : Liquid

Shelf life : 2 year (Concentrate)

Dilution Ratio : 1-10%

Genex Hygiene(A unit of aGenx Hygiene Pvt Ltd)

5B/131, Mittal Industrial Estate, Andheri-Kurla Road, Andheri(East), Mumbai – 400059

info@genexhygiene.com | genexhygiene@gmail.com | www.genexhygiene.com